MORVABON

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MEDICAL
EQUIPMENT
DEPARTMENT

Reg. No. at IRAN Medical
Equipment department: 1855

manufactureth

Head Office: Unit 154, 7 floor, c block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819

Warning:

- ▶ Keep in a cool, dry, well-ventilated area. (5-25° c)
- ▶ Keep containers tightly closed.
- ▶ Do not use after expiration date.

Instructions for use:

TOOTH PREPARATION: Prepare tooth in usual manner. For pulp capping, use calcium hydroxide. Clean the prepared tooth with pumice and water. Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with oil-free air. **DO NOT DESICCATE.** Prepared surfaces should appear moist (glistening).

1. POWDER AND LIQUID DISPENSING

The standard powder to liquid ratio is 2 / 1g.

Mixing time:

Dispense powder and liquid onto the pad. Using the plastic spatula, add all the powder to the liquid. Mix rapidly for 30 seconds.

Note:

For larger amounts, divide the powder into two equal parts. Mix the first portion with all of liquid for 5 seconds. Incorporate the remaining portion and mix the whole thoroughly for 25 seconds (total 30 seconds).

2.

mixing time:

- ▶ 30 s
- ▶ usable time after mixing (at 23° C): 2 minutes

Setting Time from end of mixing:

- ▶ 5 minutes

Contraindication:

- ▶ Pulp capping.
- ▶ Avoid use of this product in patients with known allergies to glass ionomer cement.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates a medical device that needs to be protected from moisture.

Produced by Morvabon Co.
www.morvabon.com
info@morvabon.com
Made in IRAN

Glass Ionomer Luting Cement

self cure glass ionomer luting and lining cement

Compositions:

- ▶ powder contains: Alumino silicate 15 g
- ▶ liquid contains: Acrylic resin 10ml

Indications:

- ▶ Luting of crowns, bridges, inlays, onlays, pins, posts and orthodontic bands.

Advantages:

- ▶ Low solubility in the mouth
- ▶ Low acidity
- ▶ Biocompatible
- ▶ Easy application
- ▶ High compressive and tensile strength
- ▶ Pulp friendly
- ▶ Good adhesion to dental tissue
- ▶ User-friendly working and setting time

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Glass ionomer
- ▶ Use eye protection, mask and gloves when using Glass ionomer
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice
- ▶ For dental use only
- ▶ DO NOT mix powder or liquid with components of other glass ionomer cements.
- ▶ Eye contact: Immediately flush opened eye under running water for several minutes, consult an eye specialist.
- ▶ Skin contact: In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol.
- ▶ In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.