# MORVABON

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manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7nd floor, c block,  
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1438

Fax: 0098 21 6690 8819

Factory: Unit 1 5,136,139, 2nd floor, Seraj Bd.,  
20thkm of Damavand Rd., Tehran, Iran

Tel: 0098 21

Fax: 0098 21

### warning:

- ▶ Keep out of reach of children.
- ▶ Do not use after the expiration date.
- ▶ Store in the original container and kept.
- ▶ at room temperature.
- ▶ Protect from direct sunlight.

### Direction to use:

- ▶ For using this paste, take a sufficient amount paste put on the tooth.
- ▶ The operation of the brushing should be done first with a little round and then with a high round.
- ▶ For interception contamination paste, close the door after use it, To restore the dough Removed avoid inside the dish.
- ▶ Throughout the procedure should be avoid excessive dilution and swallowing of the tooth surface. Saliva evacuation is recommended to avoid
- ▶ excessive dilution and swallowing of the paste. This dough does not need to be diluted with cold
- ▶ water or serum.

### Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance

### Available packaging: 150 grams

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	 Indicates the Temperature Limit

# Prophy-Mor quick

Contain of micro & nano silica

**For Scrubbing curing, contains fluoride ions**

## **Ingredients:**

sodium fluoride, Soft and rough mineral silicate, pumice (micro and macro), glycerin, Nano and micro Silica, food colour additives.

## **Intended use:**

- ▶ Anti plaque, cleaner and containing Anticavity substances.
- ▶ Polishing suitable for teeth surface.
- ▶ Picked up and remove stains.

## **Advantage:**

- ▶ Easy usage by brushing the teeth
- ▶ Suitable adhesion for tooth placement during brushing
- ▶ Effective Scrubbing at the micro and Nano surface without damaging the gum and tooth
- ▶ Reducing microorganism of the mouth
- ▶ Helper as anti-decay (Containing fluoride ion)
- ▶ Having a major role for preventing gum of inflammations and gum of bleeding and gum disease caused by tooth plaque.

## **Precautions:**

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with **Prophy-mor Quick**.
- ▶ Use eye protection, mask and gloves when using **Prophy-mor Quick**.
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water.
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice.
- ▶ DO NOT inject this product.
- ▶ For dental use only.