MORVABON

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manufacturer

Pardis Morvabon Pharmed Co.:

Head Office: Unit 154, 7nd floor, c block,
Borjsaz Bd, Azadi Street, Tehran, Province, Iran

Tel: 0098 21 6658 1 38

Fax: 0098 21 6690 8819

Factory: Unit 135,136,139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran

Tel: 0098 21 7625 1702

Fax: 0098 21 7648 1457

MEDICAL
EQUIPMENT
DEPARTMENT

IFDA

Morva-Flow Composite

Light Cure Flowable Composite

Morva-Flow Composite is a light cure flowable hybrid resin that combines the properties of Flowability, strength to provide the dentist with the ability to do multiple restorative procedures with a single material.

Composition: Resin, Filler.

Uses:

- ▶ Class III, Class IV and Class V restorations
- ▶ Root carries and enamel defects
- ▶ Repair of composite or ceramics veneers
- ▶ Ideal as a base or liner to help prevent carries recurrence

Benefits:

- ▶ Fast application
- ▶ Low viscosity, high flow characteristics
- ▶ Excellent polishability
- ▶ Superior color stability
- ▶ Radiopacity
- ▶ High compressive strength

Instructions for use:

- ▶ Prepare Cavity. Wash thoroughly with water spray and air dry at maximum air pressure for 5 seconds.
- ▶ Select the desired shade.
- ▶ Dispense **Morva-Flow** onto prep with provided dispensing tip. Use provided brush to apply the composite to the entire cavity wall and beveled area with moderate pressure for 15-20 seconds to obtain a thin layer (<0.5mm). Remove excess material around margins with the brush if necessary.

Note: Replace syringe cap after each use to prevent the resin from hardening in the syringe

- ▶ Light cure for 20 seconds
- ▶ After lining the cavity wall and beveled area with **Morva-Flow**, build the restoration with more **Morva-Flow** in increments of 2mm or less.
- ▶ Light cure each increment for 20 seconds.
- ▶ Finish and polish using a polishing system.

Storage:

- ▶ It would be better to store in a well-ventilated place at refrigerated temperature between 2°C and 8°C when not in use. Allow material to reach room temperature prior to use.
- ▶ **Morva-Flow** material containers should be stored with the original cap only.
- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.
- ▶ Do not freeze.
- ▶ kept out of direct sunlight

Precautions:

- ▶ Only for dental use.
- ▶ In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice.
- ▶ Wear suitable protective clothing, eyewear, and gloves.
- ▶ Is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins.
- ▶ Use of a rubber dam or adequate isolation is recommended.
- ▶ The container should be tightly closed immediately after use.
- ▶ Use only in well ventilated areas.
- ▶ Avoid skin and eye contact to prevent irritation and possible allergic response. If contact with skin or eye occurs immediately remove material with cotton and wash thoroughly with water and seek medical advices.

 Consult instructions for use	 Indicates the batch code	 Catalogue number	 Indicates the date after which the medical device not to be used.	 Indicates a medical device that needs protection from light sources.	 Indicates a medical device that needs to be protected from moisture.
 Do not use if Package is Damaged	 Indicates the medical device manufacturer (Name & Address)	 Indicates a medical device that has not been subjected to a sterilization process.	 Indicates the device intended for use by professional person only.	 Indicates that the medical device can cause discomfort and redness, irritation, or blistering of the skin	 Indicates the Temperature Limit